

SOLUZIONE DI LAVAGGIO EXTRA EXTRA WASHING SOLUTION

REF. 3972 – 2x100 ml



USO PREVISTO

La Soluzione di lavaggio EXTRA è una soluzione adatta per il lavaggio straordinario delle cuvette di lettura dei seguenti analizzatori automatici: BT 2000 / BT 3000 TARGA, BT 1000, BT 2000 PLUS, BT 3000 PLUS.^(*)

^(*)Nomi registrati dalla BIOTECNICA INSTRUMENTS S.p.A.

CARATTERISTICHE DEL PRODOTTO

La Soluzione di lavaggio Extra contiene HCl 1.0 N e tensioattivi.

PREPARAZIONE

Soluzione pronta all'uso.

PROCEDURA

Effettuare il lavaggio straordinario delle cuvette almeno una volta a settimana e qualora l'analizzatore, senza apparente causa, produca risultati non coerenti. Al termine della procedura di lavaggio, rimuovere sempre la soluzione dall'analizzatore e conservarla nel flacone ben chiuso.

CONSERVAZIONE E STABILITA'

Il prodotto è stabile fino alla data di scadenza indicata in etichetta se conservato a 2-25°C, al riparo dalla luce e in assenza di contaminazioni.

Conservare i flaconi chiusi quando non in uso.

PRECAUZIONI ED AVVERTENZE



R36/37/38 Irritante per gli occhi, le vie respiratorie e la pelle.

Utilizzare secondo la buona pratica di laboratorio, evitando il contatto con la pelle, gli occhi e l'ingestione.

SMALTIMENTO DEI RIFIUTI

Smaltire il prodotto secondo le norme locali vigenti.

INTENDED USE

The EXTRA Washing Solution is a solution suitable for extraordinary washing on reading cuvettes of following analyzers:

BT 2000 / BT 3000 TARGA, BT 1000, BT 2000 PLUS, BT 3000 PLUS.^(*)

^(*)Names registered by BIOTECNICA INSTRUMENTS S.p.A.

PRODUCTS CHARACTERISTICS

Extra Washing Solution contains HCl 1.0 N and surfactants.

PREPARATION

Solution ready to use.

PROCEDURE

Extraordinary washing must be done at least once a week and when the analyzer give non sense results.

At the end of washing procedure the solution must be removed from the Analyzer and kept tightly closed.

STORAGE AND STABILITY

The product is stable until the expiration date indicated on the label when stored at 2-25°C, protected from light and in the absence of contamination.

Keep bottles closed when not in use.

PRECAUTIONS AND WARNINGS



R36/37/38 Irritating to eyes, respiratory system and skin.

Use in accordance with good laboratory practise, avoiding contact with skin, eyes and swallow.

WASTE DISPOSAL

Dispose of product according to local laws.