

# TENSIOATTIVO CONCENTRATO CONCENTRATED SURFACTANT

REF. 3973 – 2x100 ml



---

## INTENDED USE

The Surfactant is a solution that can be used in all automated instruments to allow a better sliding of the liquid in the hydraulic circuit of the analyzer, and thus prevent the adhesion of small drops of liquid on the walls of the tubes, on the needle, on the reading cuvette.

The new formulation avoids the pollution between reagents and does not interfere with the reagents used in the tests.

---

## PRODUCTS CHARACTERISTICS

**Solution (A):** Surfactant (1g/l) with antibacterial.

**Solution (B):** HCl Solution.

---

## PREPARATION

Use each solution in the ratio: 1:1001.

Pour 10 ml of Solution (A) and 10 ml of Solution (B) in the container of the washing water of the analyzer and add 10 liters of distilled water.

You should prepare a weekly wash solution, eliminating the old residual solution and thoroughly rinse the container with distilled water.

---

## STORAGE AND STABILITY

The product is stable until the expiration date indicated on the label when stored at 2-25°C, protected from light and in the absence of contamination.

Keep bottles closed when not in use.

---

## PRECAUTIONS AND WARNINGS

The **Solution (B)** contains HCl and presents the following risks:



**R36/37/38** Irritating to eyes, respiratory system and skin.

Use in accordance with good laboratory practise, avoiding contact with skin, eyes and swallow.

---

## WASTE DISPOSAL

Dispose of product according to local laws.